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# **PURA**

## ***plus***

**Art. 249S2**

**CE**

**ITALIANO**

**ENGLISH**

**FRANÇAIS**

**DEUTSCH**

**ESPAÑOL**

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Note:

Declaration of Conformity	2
Warranty	3
Loss of the guarantee	4
Delivery terms	5
General warnings and safety rules	6
Useful contacts and addresses	6
Symbols	6
Product presentation	8
Technical specifications	8
Packaging – Transport - Storage	10
Installation	10
Demolition and dismantling	12
Machine operation and use	13
Operators instructions	14
Ordinary, Scheduled and extra maintenance	15
Diagnostics – Troubleshooting - Breakdowns	16
Supplied accessories	17
Spare parts and related documentation	18
Illustrations/diagrams	19

**DECLARATION OF CONFORMITY**

This device compliance to EMC Directive 2004/108/CE and Low-Voltage Directive 2006/95/CE.

The original declaration of conformity is provided in attached to the manual.

**WARRANTY**

- 1) **DURATION:** the product is covered by a 12-month (twelve) guarantee.
- 2) **EXCLUSION:** the guarantee excludes:
  - a) faults originating from non-performance of scheduled maintenance and due to negligence or improper use of the product by the User;
  - b) periodic checks and maintenance;
  - c) repair or replacement of parts subject to wear that are fragile or have an unpredictable life, unless they prove faulty at the time of their delivery;
  - d) faults caused by personnel or originating from technical personnel transfer and transportation;
  - e) faults or damage originating from improper use or operating errors;
  - f) faults or damage originating from contamination in the water and air infeed systems, extraordinary chemical agents or electrical events;
  - g) faults or damage originating from the use of detergents, disinfecting agents, sterilizing fluids or processes that are not described explicitly in the instruction manual for operation and maintenance;
  - h) natural color change of plastic components.
- 3) **INSTALLATION AND TESTING REPORT:** an essential prerequisite for guarantee validity is the return of the installation and testing report after all its sections have been filled in by the User and the Fitter. The report must be returned within 15 days after installation, otherwise the guarantee becomes null and void.
- 4) **LIMITS:** the guarantee entitles its holder to repairs or replacement of faulty components free of charge. The right to replace the entire machine is excluded. As for components either applied or added to TECNO-GAZ products or by third parties, having their own guarantee certificate, the conditions, limits and exclusions indicated in the relevant certificates apply.

Under no circumstances shall carrying out of one or more than one repair during the guarantee term change the guarantee expiry date.
- 5) **CLAIMS:** in the event of the Purchaser making a claim relating to the application of the guarantee and the quality or conditions of the delivered equipment, the said Purchaser shall in no way suspend and/or delay payments.
- 6) **DISPUTES:** if a dispute arises on the application and interpretation of this Certificate of guarantee, the Court of PARMA (Italy) shall be the court of jurisdiction, regardless of the location in which the purchase agreement was stipulated.
- 7) **DEROGATIONS:** any special derogations from the guarantee terms described in this section shall not imply the granting to the Purchaser of any right and shall be considered as granted for a specific case only.
- 8) **MISCELLANEOUS:** for any other issue, which is not covered by this Certificate of Guarantee, the rules described in the Civil Code and existing relevant Laws in the Republic of Italy apply.

**LOSS OF THE GUARANTEE**

The guarantee becomes null and void in the event that:

- a) the equipment presents damage caused by a fall, exposure to flames, liquid spillage, natural events, bad weather conditions or by any other cause that is not due to manufacturing defects;
- b) installation has not been performed in line with TECNO-GAZ instructions and by authorized personnel;
- c) the equipment is repaired, modified or tampered with by the Purchaser or by non-authorized third parties;
- d) when asking for an intervention under guarantee, the product serial number has been removed, deleted, counterfeited, etc.;
- e) the installation and testing report is not returned suitably filled in and signed within 15 days from installation;
- f) the Purchaser suspends and/or delays payment of amounts due, for any reason whatsoever, relating to the purchase and/or maintenance of the equipment;
- g) scheduled periodic maintenance or any other instruction envisaged in the manual for operation and maintenance is not carried out.

**DELIVERY TERMS**

1. All goods delivered must be packaged in their original packaging. If the goods are delivered in a non-genuine packaging, a charge will be applied to restore the original packaging.
2. All goods must be delivered carriage paid.
3. The client shall ask a written authorization to TECNO-GAZ sales dept. Before returning the goods by means of the "form for client's returned goods management" in compliance with the requirements of TECNO-GAZ quality management system.
4. The document accompanying the returned goods shall show:
  - a. The TECNO-GAZ product ode/item no.;
  - b. The number of the TECNO-GAZ purchase invoice or shipment document;
  - c. The product serial number and batch number;
  - d. A description of the goods returned;
  - e. The number of the form for client's returned goods management" or a copy of the same attached.
5. Returned goods are considered as accepted only after they have been examined. If they prove non-conforming, they may be sent back to the consignor at TECNO-GAZ full discretion.
6. Charges will be applied to restore and test the returned product.
7. Parts replaced under guarantee must be returned, carriage paid, to TECNO-GAZ.
8. If the parts are not returned, their cost will automatically be debited to the recipient.
9. Tecno-gaz does not accept goods returns from end users.
10. Goods sent to TECNO-GAZ for repair are managed as returned goods.

## GENERAL WARNINGS AND SAFETY RULES

-  Make sure that the equipment is supplied with the correct voltage as shown in the plate.
-  Do not remove the plate.
-  Make sure that the machine is equipped with a grounding system.
-  Use a dry cloth to clean the machine.
-  Before any operation is performed, disconnect the power cable from the socket.
-  Use original spare parts only.

*Non-compliance with the rules above relieves TECNO-GAZ of all responsibility.*

## USEFUL CONTACTS AND ADDRESSES

For technical service please apply to the dealer that invoiced the device, or directly to:

TECNO-GAZ SpA

Str. Cavalli n°4 – Sala Baganza –Parma – ITALIA. Tel. +39 521 8380 - Fax +39 521 833391

E-Mail : [info@tecnogaz.com](mailto:info@tecnogaz.com)

<http://www.tecnogaz.com>

## SYMBOLS

Tecno-Gaz S.p.A. – Strada Cavalli, 4, 43038, Sala Baganza, Parma

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

General warning and information for the user.

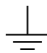
Pay attention to the instructions provided alongside this symbol.

Conformity with the Directive 2004/108/EEC and Directive 2009/95/EEC and further amendments.

PE

Equipotential point.

	Protection GROUNDING SYSTEM.
	(Operating) GROUNDING SYSTEM.
~	Alternate current.
	Warning: high voltage. Risk of electrocution!
	Read the instruction manual carefully.
	Applied part.
<b>POS.</b>	Position (number identifying a component in the figures or diagrams).
<b>Fg.</b>	Figure.
$\Omega$	Ohm (Electric Resistance unit).
<b>s</b>	Seconds (time unit).
<b>W</b>	Watt (Power unit).
<b>Hz</b>	Hertz (Frequency unit).
<b>mm</b>	Millimeters (Length unit).
<b>A</b>	Ampere (Electric Current unit).
<b>V</b>	Volt (Electric Voltage unit).
<b>Pa</b>	Pascal (Atmospheric Pressure unit).
<b>°C</b>	Degrees Centigrade (Temperature unit).
<b>kg</b>	Kilograms (Weight unit).

**PRODUCT PRESENTATION**

**PURA PLUS™** is a demineralizer, an accessory to TECNO-GAZ S.p.A. autoclaves. This product is used to remove mineral salts from the water coming from the water main to feed the autoclave.

 **USING NON-DEMINERALIZED WATER ADVERSELY AFFECTS AUTOCLAVE OPERATION.**  
**CONNECT PURA PLUS™ ONLY TO APPROPRIATE TECNO-GAZ AUTOCLAVES.**

NOTE: IN ORDER TO CONNECT **PURA PLUS™** TO AUTOCLAVES, REFER TO THE INSTRUCTIONS IN THE AUTOCLAVE MANUAL AND SEE THE PICTURE 18.

**TECHNICAL SPECIFICATIONS**

***Mechanical specifications:***

Installation room temperature	+5°C ÷ +30°C
Unit size (W, H, D)	350 mm, 300 mm, 150 mm
Package size (W, H, D)**	370 mm, 320 mm, 170 mm
Unit weight	3,5 Kg
Packed unit weight	4 Kg
Unit colour	White
Material composing the unit cabinet	Steel Fe370 – ABS

\*\* W = width H= height P = depth


***Electrical specifications:***

Supply Voltage	24 VAC±10% DC
Number of phases	1
Supply Frequency	50-60 Hz
Maximum power absorbed	10 W
Protection Type	3
Line cord	Tripolar 3x0.5mm / length: 2 mt

**Hydraulic specifications:**

Flow rate with supply pressure of 3.4 bar	12l/h (T= 20° and C=560 µS)
Maximum and minimum plumbing pressure	(2 to 7) bar
Conductivity value of the water supplied by <b>PURA PLUS™</b>	< 15 µS
Maximum installation distance between <b>PURA PLUS™</b> and the autoclave	2 m
Salts neutralized by the resins	Type organic and inorganic

Production at 20 °C and 560 µS

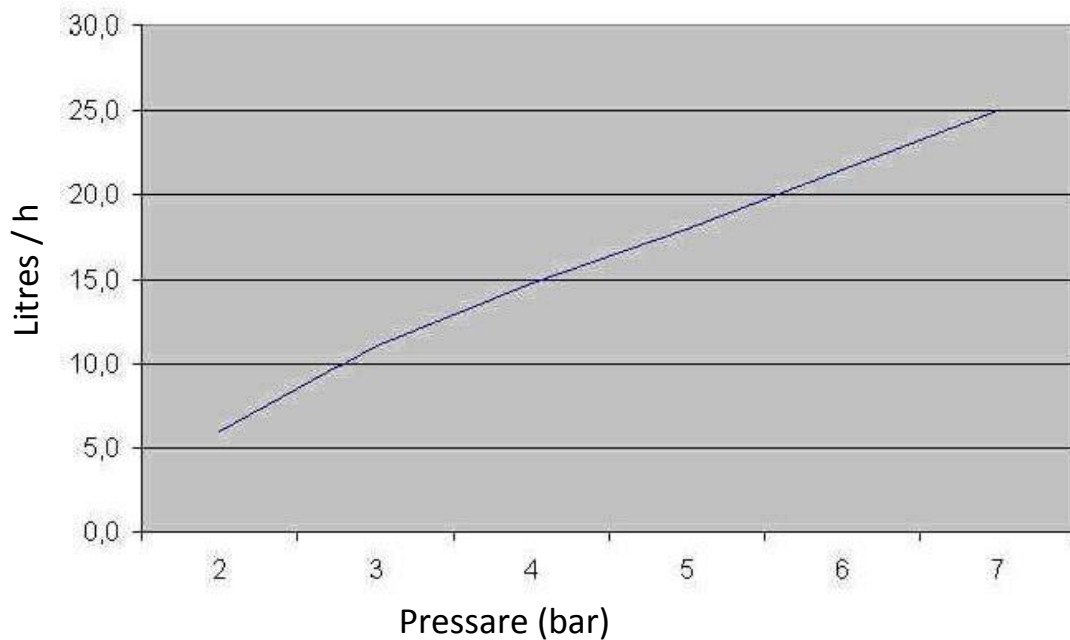**The graph refers to a supply of water with conductivity of 560 µS/cm**

**The hardness of the incoming water determines the life of the cartridge and therefore the number of litres delivered.**

**PACKAGING – TRASPOT - STORAGE**

**PURA PLUS™** is wrapped in a nylon sheet and placed in a corrugated cardboard box. Stability is guaranteed by cardboard fillers and by the accessories in the box. Transport must be done according to the regulations regarding fragile goods.

There are no particular problems with storing the product; it is possible to stack up to five boxes containing the same product and shelf life is unlimited if kept at a temperature of between 0 and 50°C.

The original packaging material must be kept during the whole guarantee period, since only with such material it is possible to transport the product without damaging it.

Failure to comply with the above shall result in termination of the guarantee.

**INSTALLATION****IMPORTANT:**

- A. DO NOT INSTALL UNDER OTHER TAPS THAT COULD WET THE WATER CLEAN.
- B. DO NOT INSTALL NEAR SOURCES OF HEAT.**
- C. DO NOT INSTALL **PURA PLUS™** OUTSIDE, FROST WOULD DESTROY THE DEVICE.
- D. THE **PURA PLUS™** INSTRUMENT SHOULD BE POSITIONED WITH ITS OUTLET FITTING AT A HIGHER LEVEL THAN THE INLET FITTING OF THE AUTOCLAVE. IF THIS IS NOT DONE, THE TAP LOCATED ON THE OUTLET FITTING WILL HAVE TO BE CLOSED DURING MAINTENANCE WORK AND WHEN CHANGING THE FILTERING CARTRIDGES, SO THAT WATER CANNOT ESCAPE.
- E. INSTALL A TAP WITH A ¾" MALE FITTING (STANDARD CONNECTION FOR WASHING MACHINES AND DISHWASHERS) ON THE MAINS WATER PIPE TO SUPPLY **PURA PLUS™**. ALTERNATIVELY YOU CAN USE THE ½" SUPPLIED CONNECTION (COD. 2ZPUA0002; SEE CHAPTER ACCESSORIES), WITH ON TAP.
- F. INSTALL A WATER DRAIN LINE FOR THE WASTE WATER PRODUCED BY **PURA PLUS™**. IF NECESSARY, USE THE COLLAR SUPPLIED WITH THE INSTRUMENT, (Code SPUA035) THAT SHOULD BE INSTALLED ON THE SINK TRAP.

NOTE 1: **PURA PLUS™** can be installed with the filters facing upwards, resting on the metal bracket equipped with feet, or hanging on the wall with the filters facing downwards or upwards, and fixed by means of the holes in the bracket.

NOTE 2: Make sure the low level led of the clean water tank of the autoclave is on; if it is not, discharge the water from the tap on the rear panel of the autoclave (see manual for use of the autoclave).

1. Position **PURA PLUS™** in the place where it is to be installed. If it is to be wall mounted, drill holes along a horizontal line spaced 254 mm (10") apart for fixing **PURA PLUS™**. Fix the instrument to the wall with the filters placed vertically and level with 2 screw anchors having a capacity of at least 5 kg.
2. Screw the pipe reduction ¾ "F-¼" pipe (Code SPUA041) on tap (INSTALLED to point E) of the water supply. An alternative you can use the fitting ½ "with its tap, supplied (Code 2ZPUA0002).
3. Connect the water supply tap to the inlet of the pre-filter (Code SPUA045) with a cut length of ¼" pipe supplied with the instrument (Code SPUA036).
4. Using another cut length of the same pipe, connect the outlet of the pre-filter to the water inlet fitting of PURA PLUS™.
5. Screw the rotating elbow fitting ¼"F – ¼"T (Code SPUA044) onto the rear water supply fitting of the autoclave (see autoclave manual).
6. Connect a cut length of ¼" pipe to the water outlet fitting of **PURA PLUS™**. *Attention: If PURA PLUS™ is installed at a lower level than the level of the autoclave inlet fitting, FOLLOW THE INSTRUCTIONS in previous POINT D.*
7. Connect the other end of the pipe referred to in point 6 to the fitting referred to in point 5, screwed onto the autoclave.
8. Connect the drain fitting of **PURA PLUS™** to the waste water line, again using a cut length of ¼" pipe.
9. Connect the power cord of PURE™ at a special jack on the rear panel of the autoclave
10. Rotate the two ball valves so that the levers are parallel to the tubes going into and out by themselves (Fig. 11, 12).
11. Open the tap water supply upstream of **PURA PLUS™** and let water run for at least 1 minute.
12. Rotate the two ball valves so that the levers are perpendicular to the tubes (Fig. 13 & 14).
13. Verify that the final condition of **PURA PLUS™** is similar to that shown in Fig. 15 & 16.
14. Verify that there is no loss of water and rearrange **PURA PLUS™** original position.
15. The settings and programming of the autoclave are given in the manual of the autoclave itself.
16. Press and hold the button on the manual water filling the front operator panel of the autoclave until a beep.
17. The setting and programming of the autoclave are shown in the autoclave manual.

**DEMOLITION AND DISMANTLING**

Directive on waste from electrical and electronic equipment (WEEE).

Pursuant to DIRECTIVE 2002/96/EC, this symbol shows that the product must not be disposed of as urban waste at the end of its operating life.

The product must be taken to centers specialized in separate collection of electric and electronic equipment or to a dealer, when a similar product is purchased.

The machine holder shall be responsible for bringing it to collection centers.

For further information about collection systems, address the local waste disposal service.

**Correct disposal of the equipment, which is no longer used, prevents negative consequences for the environment and human health.**

**Abuses will be prosecuted by law.**

**DISPOSAL OF SPENT FILTERING CARTRIDGES:**

These products, if disposed of unused and uncontaminated should be treated as special non-hazardous waste in accordance with EC Directive 91/689/EEC. The disposal procedures must observe all the national and local laws and regulations governing hazardous waste. For the spent, contaminated product and for its residues, the relevant EWC codes are indicated below:

Description	EWC code
Sediment cartridge	190901
Carbon block cartridge	190904
Resin cartridge	190905
Membrane TFC 75 GPD	190901

**SPENT CARTRIDGES MUST BE PLACED INSIDE CELLOPHANE BAGS AND SEALED.**

## MACHINE OPERATION AND USE

**PURA PLUS™** IS INTENDED ONLY FOR FEEDING TECNO-GAZ AUTOCLAVES WITH PROVISION FOR THIS.

ONCE INSTALLED, THE WATER CLEAN WILL START OPERATING ONLY WHEN THE AUTOCLAVE CALLS FOR WATER.

USE FOR OTHER PURPOSES IS NOT COVERED BY THE GUARANTEE; TECNO-GAZ WILL NOT BE HELD LIABLE FOR DAMAGE.

The **PURA PLUS™** is comprised of the following:

- A solenoid valve operated by the autoclave, which open and closed the flow of water upstream of a set of two filters that precede the core of the system, i.e. the osmotic membrane which is installed in its own specific container
- At the out feed of this, there are two water flows: the permeate, which consists of already highly demineralised water (salt removal >95%), and the concentrate, which is then conveyed to the drain.
- Following the line of the permeate, a resin cartridge further demineralises the water which flows through it, neutralizing the residual salts.
- Two lever ball valves allow water to be flushed through the filters, independently of autoclave operation, in order to wash them during replacement of the same.

When the autoclave calls for water, the solenoid valve opens and the water flows into the autoclave tank.

When the maximum level has been reached in the tank, the autoclave stops its request for water and the whole system switches off.

### OPERATING TEST

To check that the device works, after finishing the installation operations, switch on at the autoclave master switch and activate the program for water loading from the **PURA PLUS™** (see instructions given in the autoclave manual).

If the led that indicates the max level of water in the autoclave tank is off, the **PURA PLUS™** will call for water. When the max level is reached, the related led of the autoclave comes on, delivery of water stops and the green led of the **PURA PLUS™** goes out. If the autoclave tank is at the max level, the water must be drained out by opening the tap at the back of the machine in order to check the operation of the **PURA PLUS™**.

When the max level led of the autoclave goes out, delivery of water from the **PURA PLUS™** starts and the green led comes on. Delivery of water from the **PURA PLUS™** continues until the max level is reached.

**OPERATORS ISTRUCTIONS**

- CONNECT **PURA PLUS™** ONLY TO AUTOCLAVES COMPATIBLE WITH THAT UNIT.
- ALWAYS INSTALL A TAP UPSTREAM OF THE DEMINERALIZATION CIRCUIT IN ORDER TO AVOID FLOODS;  
AT THE END OF THE DAY OR WHEN THE AUTOCLAVE IS NOT GOING TO BE USED FOR SOME DAYS, CLOSE THE TAP INSTALLED UPSTREAM OF **PURA PLUS™**
- CONNECTING **PURA PLUS™** TO THE HOT WATER LINE IS STRICTLY FORBIDDEN!
- **PURA PLUS™** CAN BE MOUNTED TO THE WALL OR CAN BE SUPPORTED ON A SURFACE.
- THERE IS NO MINIMUM CAPACITY. THE MAXIMUM CAPACITY IS 12 l/h
- MAXIMUM AND MINIMUM PRESSURE FOR WATER CLEAN OPERATION:  
from 2 to 7 bar
- USE OF **PURA PLUS™** IS GRENTEED WITH MAX HARDNESS OF THE INCOMING WATER OF 200 °F.
- OUTPUT CONDUCTIVITY VALUE: < 15 µS (Micro Siemens); THE INPUT HARDNESS IS IRRELEVANT.
- THE MEMBRANE AND RESINS NEUTRALIZE ALL KINDS OF ORGANIC AND INORGANIC SALT.
- THE CARTRIDGES MUST BE HANDLED WITH NORMAL PRECAUTIONS. THE USE OF PROTECTIVE GLOVES IS MANDATORY
- THE FILTERING CARTRIDGES MUST BE DISPOSED OF AS SPECIAL WASTE (SEE SEC.7)
- INSTALL **PURA PLUS™** WITHIN 2 METRE OF THE AUTOCLAVE
- DO NOT EXPOSE **PURA PLUS™** THE WATER CLEAN TO FROST
- DO NOT INSTALL NEAR SOURCES OF HEAT
- DO NOT INSTALL UNDER OTHER TAPS

**ORDINARY, SCHEDULED AND EXTRA MAINTENANCE**

## ROUTINE MAINTENANCE:

Maintenance of **PURA PLUS™** consists simply of changing the filtering cartridges of the resin container, which is to be carried out once a year.

## CHANGING FILTERS AND RESINS (Fig. 19):

1. Close the water tap upstream of **PURA PLUS™**.
2. If **PURA PLUS™** is installed below the level of the autoclave, close the tap on the back of the autoclave to avoid emptying the tank of the autoclave and the consequent flooding.
3. Position **PURA PLUS™** on a flat surface, unscrew and remove the sediment pre-filter and the carbon block filter (Fig. 1 & 2). Mop up the small amount of water that comes out with paper or rags.
4. Screw on the new filters, making sure you put each filter in the right position (the carbon block filter is fitted on the side, Fig. 3 & 4). Screw them on tightly but do not over-tighten.
5. Remove the safety clips from the fittings of the resin container (Fig. 5) and take out the connecting pipes by pressing the retaining ring towards the body of the fitting and then pulling on the pipe (Fig. 6).
6. Remove the spent resin container and replace it with a new one, taking care to keep it the same way round (Fig. 7).
7. Insert the pipes in the fittings, press them home (Fig. 8) and insert the safety clips (Fig. 9 & 10).
8. Turn the two ball valves so that the levers are parallel to the pipes entering and leaving them (Fig. 11 & 12).
9. Open the water tap upstream of **PURA PLUS™** and let the water run for at least 1 minute.
10. Turn the two ball valves so that the levers are at right angles to the pipes (Fig. 13 & 14).
11. Check that the final condition of **PURA PLUS™** resembles the one shown in figures 15 & 16.
12. Record label of the sediment filter the date of filters exchange of **PURA PLUS™** (Fig. 16 b).
13. Check that no water is leaking out and put **PURA PLUS™** in its original position.
14. Press and hold the button, load water manually on the control panel of the autoclave until a beep.

**ATTENTION:**

**Replacing the filters and resins is absolutely necessary after the signal from the connected autoclave, or IN ANY CASE AFTER 12 MONTHS so as not to compromise the functionality of the system and also to preserve the osmotic membrane.**

**DIAGNOSTICS – TROUBLESHOOTING - BREAKDOWNS**

TROUBLESHOOTING TABLE

ATTENTION: WITH AUTOCLAVE SWITCH SWITCHED ON

<i>Anomaly</i>	<i>Cause</i>	<i>Normal solution (possibly adopted by the user)</i>	<i>Solution by TECNO-GAZ authorized technicians</i>
<b>Ⓐ</b> <b>PURA PLUS™</b> produces no water	1 <sup>st</sup> Autoclave is not requesting water	Check that the max level led of the autoclave is off (see PAR 8).	-
	2 <sup>nd</sup> Supply tap closed	Check that the tap supplying <b>PURA PLUS™</b> is not closed by mistake.	-
	3 <sup>rd</sup> No power	Check that you have inserted the connector correctly into the autoclave	Check that the 24VAC on the contact 1 e 2, in the autoclave, is present.
<b>Ⓑ</b> <b>PURA PLUS™</b> produces very little water	1 <sup>st</sup> Low waterworks pressure	Check that the waterworks pressure is not below 2 bar.	Find the cause of the low pressure and eliminate it.
	2 <sup>nd</sup> Defective or spent osmotic membrane	NA	Replace the osmotic membrane.


**The examples mentioned above are typical anomalies that may occur. Everything outside the third column pertains to Tecno-Gaz S.p.a.**

**Contact the Service centre.**

SUPPLIED ACCESSORIES

COD. SPUA035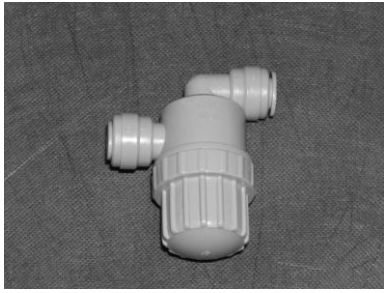


COD. SPUA045

COD. SPUA041

COD. 2ZPUA0002

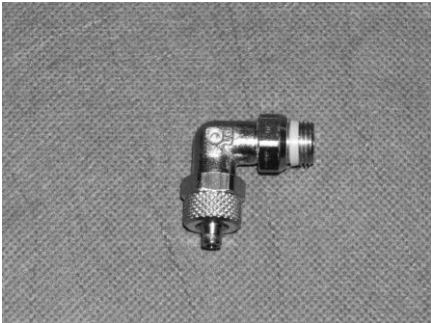COD. SPUA044

COD. SPUA036

**SPARE PARTS AND RELATED DOCUMENTATION**

<i>Code</i>	<i>Description</i>	<i>Quantity</i>
SPUA010	Female connector 5.5 mm	1
SPUA011	Connector lock nut	1
SPUA012	Membrane TFC 75 GPD	1
SPUA013	Clip 1/4"	6
SPUA014	Membrane vessel	1
SPUA015	Membrane vessel lid	1
SPUA016	Membrane vessel gasket	2
SPUA017	Rubber foot	4
SPUA018	Painted steel support bracket	1
SPUA019	Single clip 60 mm	2
SPUA021	Double clip 50x60 mm	2
SPUA026	Filter housing	2
SPUA028	Straight fitting 1/4"x1/4" pipe	2
SPUA029	Elbow fitting 1/8"x1/4" pipe	2
SPUA030	Elbow fitting 1/4"x1/4" pipe	1
SPUA031	T fitting 1/4" side	2
SPUA032	T fitting 1/4" pipe-pipe-pipe	2
SPUA033	Nipples 1/4"x1/4"	1
SPUA034	Flow restrictor 400 ml	1
SPUA035	Drain flange 1/4"T x 40mm	1
SPUA036	Pipe 1/4" 8 metres	1
SPUA170	Solenoid valve 24V DC 1/4"Fx1/4"F	1
SPUA038	Check valve with spring 1/8"	1
SPUA039	Quick-connect tap 1/4"	2
SPUA041	Reducer 3/8"F - 1/4"F	1
SPUA042	Check valve 1/4"	1
SPUA043	Pipe core 1/4"	17
SPUA044	Rotating elbow 1/4"M - 1/4"T	1
SPUA045	Mesh pre-filter 14"T - 14"T	1
SPUA169	Rubber grommet	1
SPUA171	Flame resistant electric cable 3x0, 5 mm (2 meters)	1
SPUA172	Clamp connection	1
SPUA173	Binder 4-pin male connector 90°	1
2ZPUA0002	Total fitting connection ½ "	1
SPUA025	Carbon Block Filter ****	1
SPUA023	Sediment filter ****	1
SPUA022	Resin cartridge 0.3l with the couplings ****	1

**THE CARTRIDGES (SEDIMENT FILTER, CARBON FILTER AND RESINS FROM LT. 0.300 HOUSING WITH PLUGS) ARE SUPPLIED ONLY IN A UNIQUE KIT:**

Cod. 248-S-2

ILLUSTRATIONS/DIAGRAMS

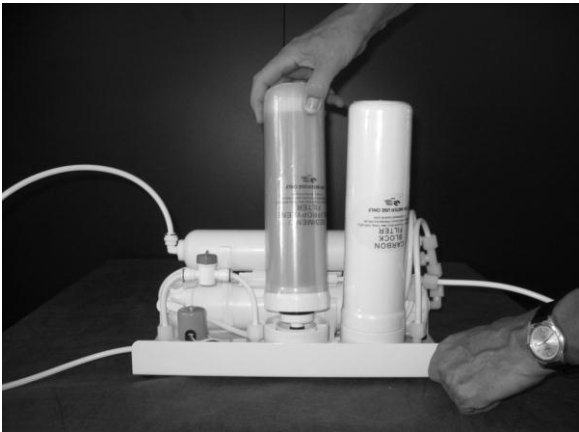
Fig. 1

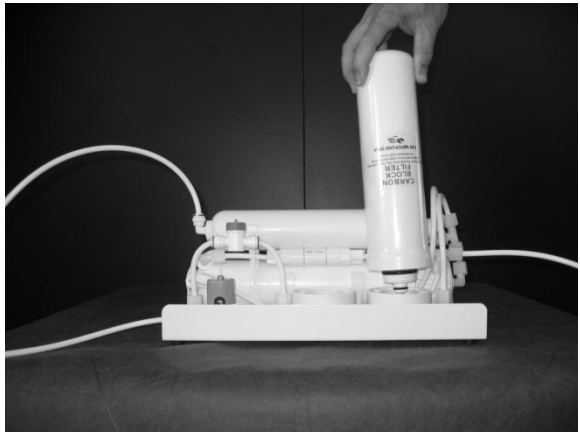
Fig. 2

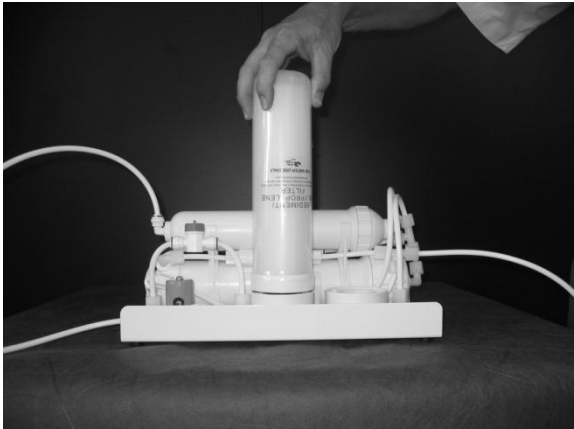
Fig. 3

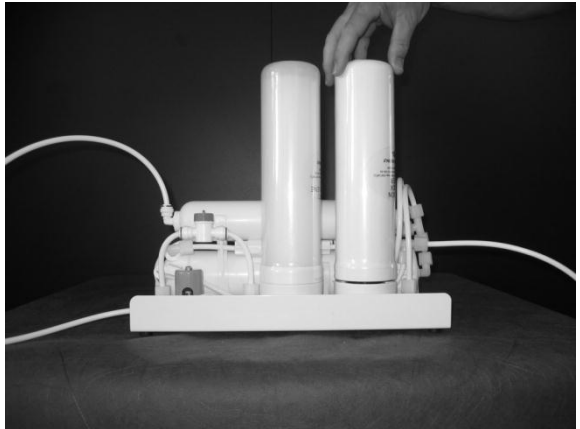
Fig. 4

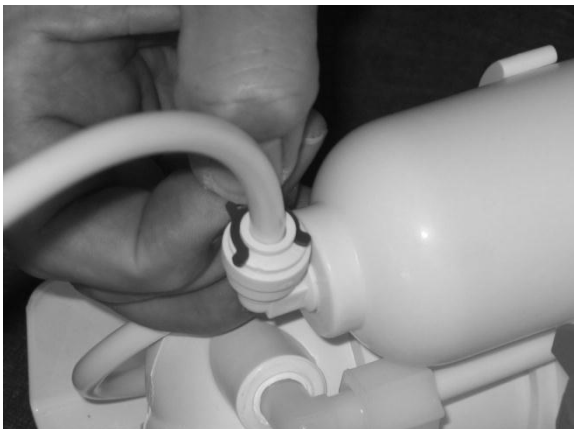
Fig. 5

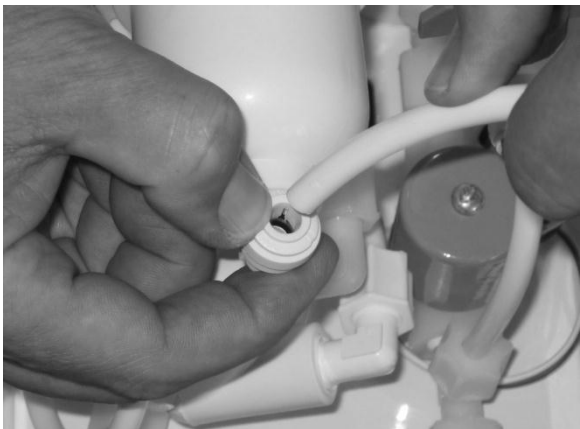
Fig. 6

Fig. 7

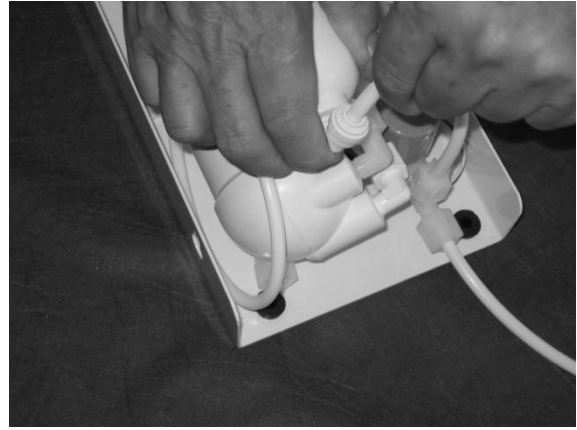
Fig. 8

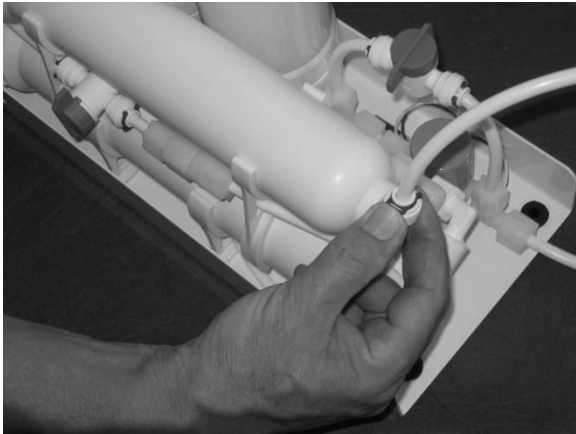
Fig. 9

Fig. 10

Fig. 11

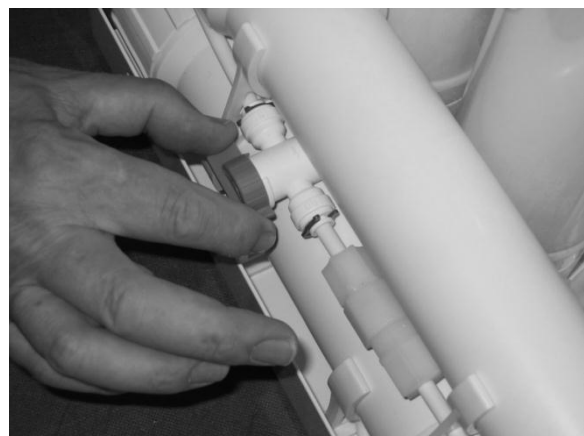
Fig. 12

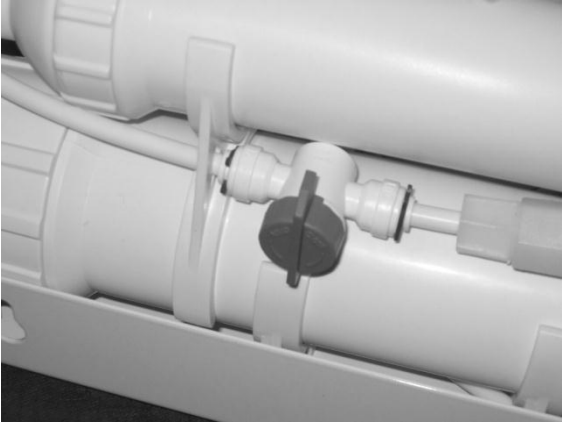
Fig. 13

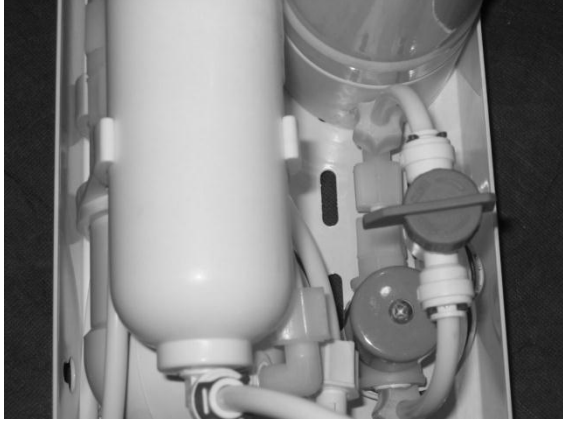
Fig. 14

Fig. 15


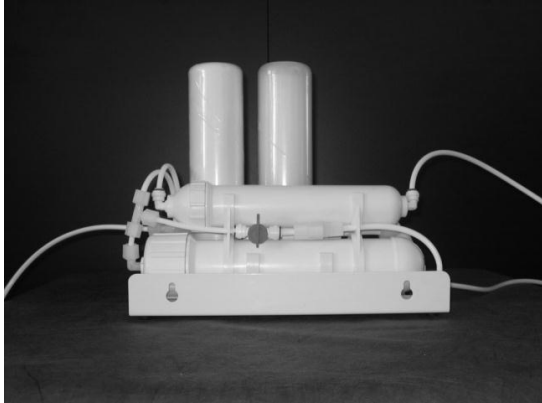
Fig. 16

Fig. 16 b

**POSITION OF PURA PLUS™'S FITTINGS**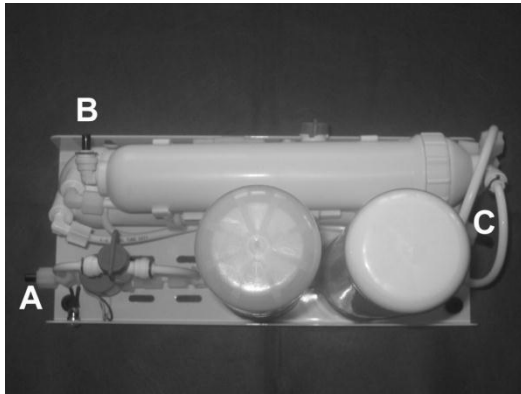

- A: Raw water inlet
  - B: Demineralised water outlet
  - C: Waste water outlet
- Fig. 17

*In order to connect Pura Plus™ to autoclave:*

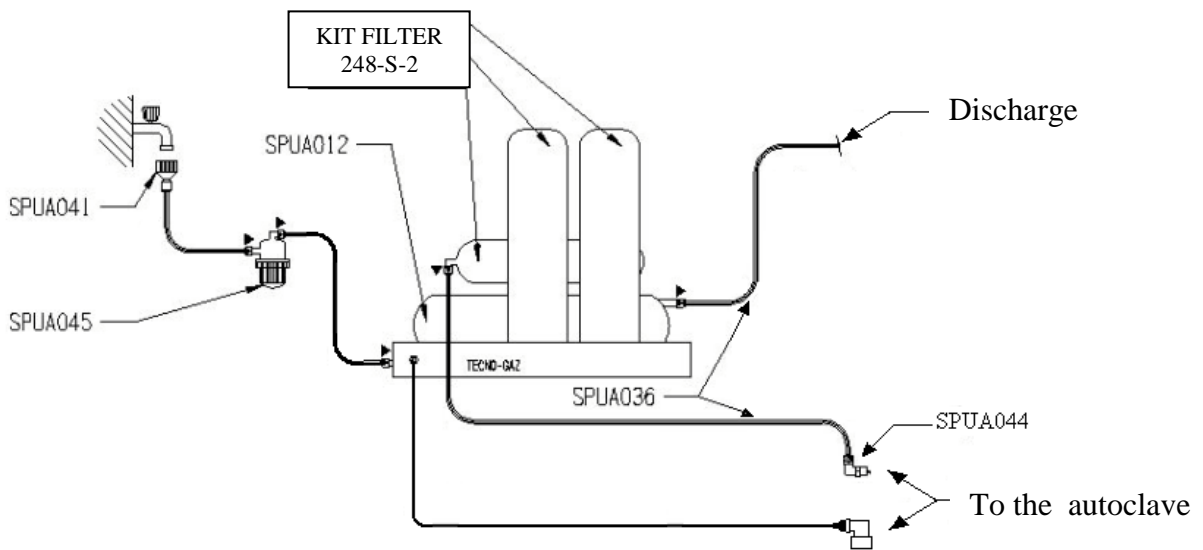
Fig. 18

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